DEUTSCH

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Table with 2 columns: Symbol, Description. Includes symbols for Caution, Instructions, Application, Disposal, Manufacturer, Storage/Transport, Operating, and CE marking.

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ENGLISH

Signs and symbols
The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

Table with 2 columns: Symbol, Description. Includes symbols for Caution, Instructions, Type BF applied part, Disposal, Manufacturer, Storage/Transport, Operating, and CE marking.

Included in delivery: • Thermometer • Instruction for Use • 1 x 1.5V LR41 battery
Important safety information – retain for later use.
In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

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Table with 2 columns: Symbol, Description. Includes symbols for Attention, Instructions, Apparatus type BF, Elimination of residues, Manufacturer, Packaging, Protection, Designation, Storage/Transport, Operating, and CE marking.

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Table with 2 columns: Symbol, Description. Includes symbols for Attention, Instructions, Application, Disposal, Manufacturer, Storage/Transport, Operating, and CE marking.

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– part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN60601-1-2 (In accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 and IEC 61000-4-4), and is subject to particular precautions with regard to electromagnetic compatibility.
Notes on electromagnetic compatibility
• The device is suitable for use in all environments listed in these instructions for use, including domestic environments.

Measurement in the mouth cavity (oral)
For this purpose, guide the tip of the thermometer carefully into one of the heat pouches beneath the tongue, to the left or the right of the root of the tongue.
Measurement of temperature in the armpit (axillary)
This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

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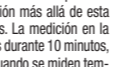
Notes on handling batteries
• If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
• Choking hazard! Small children may swallow and choke on batteries.
• Observe the plus (+) and minus (-) polarity signs.
• If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
• Protect the batteries from excessive heat.
• Risk of explosion! Never throw batteries into a fire.
• Do not charge or short-circuit batteries.
• If the device is not to be used for a long period, take the batteries out of the battery compartment.
• Use identical or equivalent battery types only.
• Always replace all batteries at the same time.
• Do not use rechargeable batteries.
• Do not disassemble, split or crush the batteries.
Cleaning/Disinfection
The device must be cleaned and disinfected before and after each use. The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties.
Name of the agent: Isopropanol 70% Pharmacy; Gigasept FF; Lysosformin; Aseptisol.
Storage/Disposal
When you are not using the thermometer, please retain it in the original protective cover.

Notes on handling batteries
• If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
• Choking hazard! Small children may swallow and choke on batteries.
• Observe the plus (+) and minus (-) polarity signs.
• If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
• Protect the batteries from excessive heat.
• Risk of explosion! Never throw batteries into a fire.
• Do not charge or short-circuit batteries.
• If the device is not to be used for a long period, take the batteries out of the battery compartment.
• Use identical or equivalent battery types only.
• Always replace all batteries at the same time.
• Do not use rechargeable batteries.
• Do not disassemble, split or crush the batteries.
Cleaning/Disinfection
The device must be cleaned and disinfected before and after each use. The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties.
Name of the agent: Isopropanol 70% Pharmacy; Gigasept FF; Lysosformin; Aseptisol.
Storage/Disposal
When you are not using the thermometer, please retain it in the original protective cover.

Table with 2 columns: Item, Value. Includes fields for Typ, Range of measurement, Accuracy of measurement, Storage temperature, Ambient temperature, and technical specifications for accuracy and humidity.

The following warranty terms shall apply:
1. The warranty period for BEURER products is either 5 years or -if longer- the country specific warranty period from date of Purchase.
In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages caused by a improper treatment, e.g. nonobservance of the user instructions.
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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Table with 2 columns: Item, Value. Includes fields for Typ, Plage de mesure, Précision, Température ambiante, and technical specifications for accuracy and humidity.

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Table with 2 columns: Item, Value. Includes fields for Tipo, Alcance de medida, Precisión, Temperatura ambiental, and technical specifications for accuracy and humidity.

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