T-Fit 270 CALL

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ITALIANO

USER MANUAL

ENGLISH

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Please read the manual before use.

- The information contained in this document will not be modified or extended in accordance with any notice.
- The watch must be charged at least 2 hours before use.

1. Product Specifications

- Model: T-Fit 270 CALL
- CPU MTK2502 ARMv7
- Memory: RAM 32Mb + ROM 32Mb
- Capacitive Touch screen 1.69" 240x480 pixel IPS
- Bluetooth version 4.0
- Functions: Heart rate detection, Blood pressure monitor, ECG, Oxygen, Sleep monitor, Sedentary, Remote Camera, Control music, Weather, 6.Multi-sport functions (Walking, Running, Hiking, Cycling, Skipping, Badminton, Basketball, PingPong, Football, Pedometer, Calorie Consumption, Distance Calculation).
- Ion Lithio Battery 3.7V/300mAh

2. Charging instruction

This product adopts magnetic charge. There are metal contacts on the back of the watch, bring the magnetic clip of the charging cable close to the metal contacts, this will tend to align correctly automatically. Connect the USB socket of the charging cable to a computer or to a 5V  1A power supply with a USB port. It is possible to view the charge status on the display.

Warning: Do not make the magnetic charging cable any 4 pins connect with conducting material at same time, it may cause short circuit.

3. Product details

3.1 Power ON/ OFF

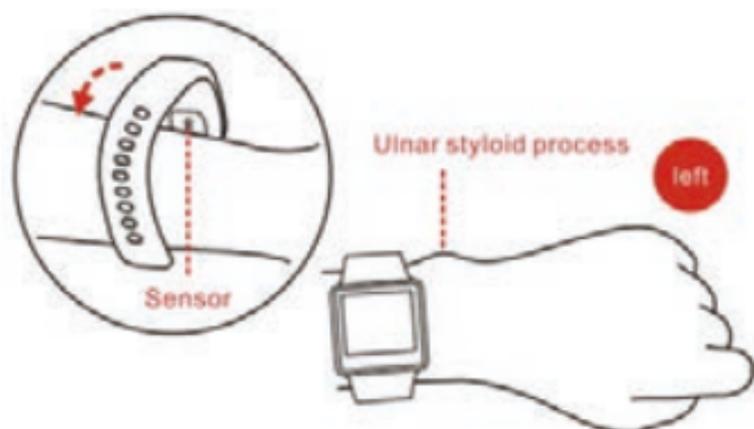
- (1) Power ON: with clock OFF, press and hold the side button for 3 seconds. After startup, the time will be displayed.
- (2) Power OFF: with watch ON, hold down the side button for 3 seconds, the watch will vibrate and then turn off.

3.2 Product description

3.3 How to wear the watch

- Wear the watch after the ulnar styloid.

- Fasten the strap to your wrist by adjusting it perfectly to the size of your wrist.
- Keep the sensor as close to the skin, avoid any movement and that external light influences the accuracy of the tests.
- Keep the test area clean, sweat or stains will affect the test results.

3.3 Changing the dial

Once the clock is turned on, long press to set the clock interface, if you want to change it, and choose from the pre-loaded ones.

4. Quick product information

4.1 Touch dial description

(1) Drag to the left to select the function page, then touch the icon to access the corresponding function.

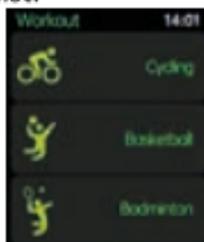
T-Fit 270 CALL

English

(2) Drag right to select the sport in the list.

(3) Drag down to view information.

(4) Drag upwards to view notifications.

(5) When the clock is on, press the side button to light up the screen, press it again to turn off the light, in navigation press it to return to the previous page.

5. APP Download and use

5.1 APP download and installation

- (1) When the watch is on, drag down and select the icon depicting the QR code , the QR code of the app will appear on the display;
- (2) Scan the QR code with your smartphone and download the "Fundo" APP, or use the following QR code:

Note: Your smartphone must support Android 4.4 or iOS 8.0 or above and Bluetooth version 4.0 or above.

5.2 APP pairing with Android and iOS phones

- (1) From the main clock screen, drag to the left and select the "Settings" icon ;
- (2) Select the item "BT Setting" and then tap "Power Off" to activate the Bluetooth function, after activation the message "Power On" will appear;
- (3) Activate the Bluetooth function on your Smartphone and open the "Fundo" APP;
- (4) From its main page, select the "Other" or "More" item in the lower right corner;

- (5) Select "Add device", wait a few seconds and from the list of devices select "TFit270CALL_LE";
- (6) After pairing, the icon  on the information page will turn green.

Pairing for managing calls, SMS and contacts:

- (7) Open the Bluetooth page of your smartphone and start searching for devices;
- (8) At the end of the search, select the "TFit270CALL" device and confirm the next connection window. After pairing, the word "Connected for calls" will appear under the TFit270CALL writing;
- (9) After pairing, the icon  on information page will turn yellow.
- (10) From the main clock screen, drag to the left and select the icon ;
- (11) Select the "Sync with phone" item to synchronize the phone contacts with the watch.

Note on the color of the icon

- **White color with red bar:** not connected
- **Blue color:** connected to the telephone for incoming/ make a calls
- **Green color:** connected with the Fundo App
- **Yellow color:** connected to the Fundo App and to the telephone for incoming/ make a calls

6. How to set up notification

In the phone settings, select "App management" or "APP", select the "Fundo" app.

Select the item "permissions" allow and activate all the items in the list.

Still in the "Fundo" app, select the "Notifications" item, also in this case allow and activate all the items in the list.

**Notice:**

Please do not close the Bluetooth notification service or terminate the background application software. It will affect the sync function between watch and phone if you close it.

7. Introduction to the main functions**7.1.1 Telephone**

Contacts: can synchronize mobile phone contacts, view and dial them via Bluetooth.

Dial: connects to the mobile phone to dial a number.

Call log: can synchronize the call log (made, received and missed).

Notifications: Synchronize all mobile messages.

7.1.2  Sleep Monitor

(1) The clock will show the user's sleep hours of the night before. The sleep measurement time is from 21:00 to 09:00 the next day. The total hours of sleep performed are displayed, divided into deep sleep and light sleep.

(2) Activation: after selecting the icon, drag your finger from bottom to top; activate the function (if deactivated), you can also view the history of the recorded data.

7.1.3 Heart Rate

The watch will measure the user's heart rate in its measurement interface. Your instant heart rate will be displayed. If the value is not detected, the zero value will always be displayed.

7.1.4 Blood Pressure

The watch will measure the user's blood pressure in its own measurement interface. It will take a few seconds for the pressure to be detected, after which the result will be displayed. If the value is not detected, the zero value will always be displayed.

7.1.5 Oxygen

The watch will measure the user's blood saturation in its measurement interface. Once its icon has been selected, select the word "Start" to start the detection, it will take a few seconds to detect the data, at the end of which the result will be displayed.

7.1.6 ECG

The watch will detect the user's ECG in its measurement interface. The instant ECG will be displayed. If the value is not detected, the zero value will always be displayed.

7.1.7 CRemote Camera

To take a picture using the watch as a shutter command, the phone must be unlocked, the "Fundo" App is active and the "Take a picture" command on the app must be active.

7.1.8 Music

Once connected to Bluetooth, it is possible to act on the playback/ pause, track skip forward

and back, music tracks on the phone and volume adjustment controls.

7.1.9 Find Phone

Once its icon has been selected, touch the word “Start”, the phone will begin to ring. After finding it, tap “Finish” to stop the search.

7.2.0 Sedentary

You can set the time period by yourself (at 30-minute intervals, minimum 0m←→300m maximum) when the sedentary time is reached, the watch will ring and vibrate to remind you to get up.

7.2.1 Sport

Sports mode: walking, running, mountaineering, cycling, basketball, badminton, table tennis, football, settings, historical records.

7.2.2 Pedometer

Before use, access the pedometer menu. Select the icon and drag your finger from bottom to top; activate the function (if disabled) and set all the personal information required. Wear the watch on your wrist, swing your arm normally while walking, the pedometer will count the steps and combine the total number of steps by calculating the calories burned.

7.2.3 Alarm

Customize the alarm time, number of rings, ring tones, and reminder types

7.2.4 stopwatch

Click the button in the lower left corner to start timing. In this interface, click the button in the lower right corner to continue timing, click the button in the lower left corner to pause timing, and slide the screen up and down to view historical data; when the stopwatch is paused, click the button in the lower right corner to clear data

7.2.5 BT Call

On the watch side, click Bluetooth search device, select the Bluetooth name of the mobile phone that you want to pair with, click pair, and the phone end/ watch side click agree to pair successfully

7.2.6 Language

Click on the icon to select and change the clock language.

8. Settings

- BT Setting: to enable/ disable the Bluetooth function and to enable/ disable the visibility of the watch.
- Clock: to select the dial, to activate/ deactivate automatic synchronization (if the date, time and format setting is deactivated).
- New notification Bright: enable/ disable.
- Sound: to set the type of alert, ringtone and notification tone.
- Volume: volume setting for Multimedia, ringtone and notifications.
- Display: to set the display type of the icons, the brightness and the shutdown time of the

display.

- Units: to set the metric system.
- Reset: reset the clock to factory settings.

Note: All data will be deleted.

- About: shows all the software information of the watch.

9. WARNING

Always consult your doctor before starting a exercise program. TREVI T-Fit 270 CALL is not a device medical device but a device capable of measuring and displaying the wearer's heart rate, blood pressure, blood oxygenation and ECG on an LCD display. The measurement results of this product are for reference only and are not intended for any medical use.

Warnings

This device is an electronic instrument of high precision; therefore, avoid using it in the following cases:

- Close to strong heat sources such as heaters and stoves.
- In environments that are too cold or too hot or very dusty.
- The appliance must not be exposed to dripping or splashing water. No liquid-filled objects, such as vases, must be placed on the appliance.
- If liquids penetrate into the device, bring the device to the nearest authorized TREVI service Centre.
- Do not use the device near flammable or explosive gases, this may cause a malfunction of the device or a fire hazard.

T-Fit 270 CALL

English

- No naked flame sources, such as lighted candles, must be placed on the appliance.
- Keep the device out of reach of children. The device is not a toy. The device is made up of small removable parts which, if ingested, can cause suffocation.
- Do not expose the device to strong shocks.
- Store the product in a dry place when you are not using it.
- Keep this instruction manual.

Precautions for correct disposal of the product

The symbol shown on the equipment indicates that waste must be disposed of in “separate collection” and so the product must not be disposed of together with urban waste.

The user must take the product to special “separate waste collection centres” provided by local government, or deliver it to the retailer against the purchase of a new product.

Separate collection of waste and subsequent treatment, recycling and disposal operations promotes the production of equipment with recycled materials and limits negative effects on the environment and on health caused by improper handling of waste.

Illegal disposal of the product leads to the enforcement of administrative penalties.

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<https://www.trevi.it/dms/zidi/TREVI/DOCTFIT270CALL.pdf>

Simplified EU Declaration Of Conformity

Hereby TREVI S.p.A. declares that the equipment
T-Fit 270 CALL

is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following
internet address

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For additional information and updates of this product see: www.trevi.it

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Il fabbricante TREVI S.p.A. dichiara che il tipo di apparecchiatura